

Treatment Plan Legend

Patient name

Elyzabyth Lemon-Merenguez

Case number · Plan type · Date

291585 · Unlimited · April 15, 2019


Treatment plan steps



Treatment icons



Case revision

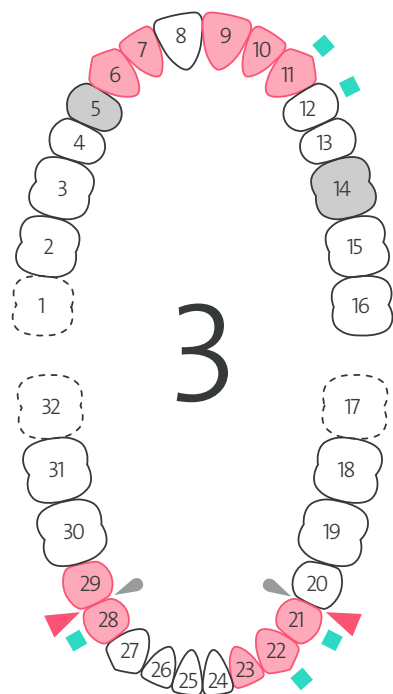

A pink tooth is a tooth that is moving.

A grey tooth is a pontic.

A dotted tooth is a missing tooth.

The square icon is for engagers. It indicates where you should place an engager.

The triangle points to the teeth you should perform IPR between. There are different size triangles to represent the IPR size.

The teardrop icon is for closed contacts.

The 'x' icon shows where engagers are and that they need to be removed.