


# How to install engagers

Recommended use instructions for ClearCorrect engager templates.

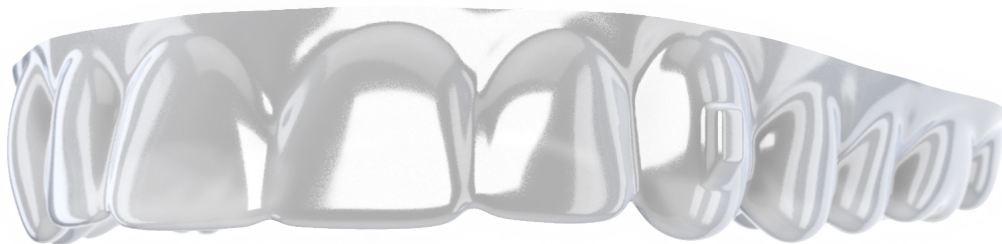

Engager template



**\*Suggested compatible composites for the copolyester Engager Template:** Tetric EvoCream and Tetric EvoFlow (Ivoclar Vivadent); Filtek Z250, Filtek Supreme Flowable and Filtek Supreme XTE (3M ESPE); Venus Diamond and Venus Pearl (Heraeus-Kulzer), and SonicFill 3 (Kerr Dental).


# How to install engagers

The ClearCorrect Engager Template is designed to make bonding engagers to the tooth surface more efficient and accurate. The thin and flexible template material allows for faster and easier removal of the template, while the Low Trimline provides enhanced retention of the engager template on teeth, ensuring a more accurate placement. Both improvements are designed to optimize chair time and streamline the orthodontic procedure of engager placement.

*The Engager Template material may not be available in all regions.*



Try on the template.

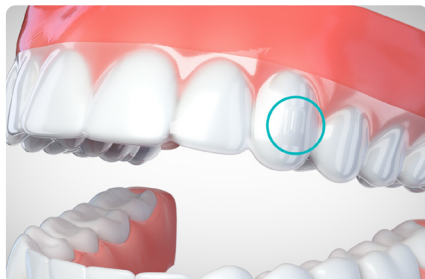


Identify the location of all engagers.



Etch tooth only in the area where engager will bond to tooth. Do not etch the whole facial tooth.



Rinse off the etching acid.

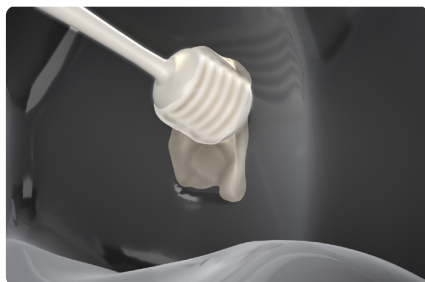


If preferred, lightly coat Engager Template cavities with petroleum jelly. (Optional)



Fill Engager Template cavities with paste-type or flowable type composite resin.




Place bonding agent / primer on each etched tooth.

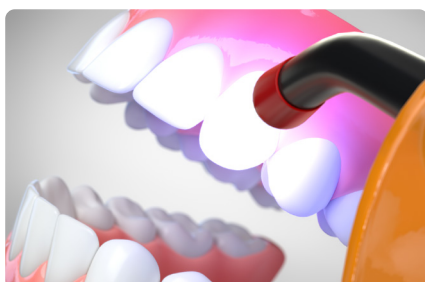

Light cure the bonding agent.



Insert Engager Template.

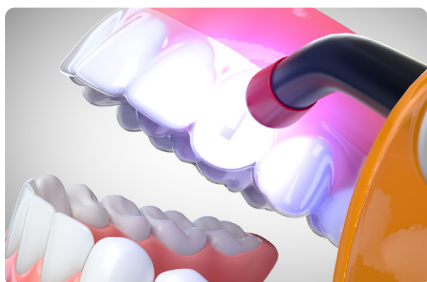

Light cure the composite.



Remove the Engager Template and clean off material flash material with scaler.



Check the fit of the aligner.